

Siltape®

Soft silicone perforated tape roll

Product description

Siltape® is a soft silicone fixation tape delivered on a handy roll with perforations for ease of use, no scissors are required.

Use

Siltape® can be used in any situation where a regular adhesive tape would be used. It is particularly useful where delicate or sensitive skin is an issue. It is ideal for use on pediatrics and neonates. Siltape® is a suitable solution to taping down eyelids. Acrylic based tapes are often used, these can be painful to remove and in some cases cause tissue damage or pull out eyelashes.

Siltape® is great for securing IV lines and fistula needles as it can be repositioned with no loss of adherence and with atraumatic soft silicone, no tissue damage will occur.

Small incision sites such as biopsies can be covered with Siltape® with no harm to the wound or surrounding skin.

Siltape® is ideal for protecting delicate tissue from pressure damage or sores from tubing; this is especially useful on the face and tops of ears.

Siltape® can be applied directly to the skin and tubing placed on top.



Features

- ✔️ Atraumatic soft silicone, reduced pain on change, adjustment or removal
- ✔️ Soft & conformable
- ✔️ Adheres only to dry skin not to a moist wound
- ✔️ Easily adjustable with no loss of adherence

Indications

- ✔️ Hold dressings in place
- ✔️ Taping down eyelids
- ✔️ Securing IV lines
- ✔️ Over small incision sites
- ✔️ Under oxygen masks on bridges of noses
- ✔️ Securing fistula needles
- ✔️ Protection from pressure damage or sores from tubing



Contra-indications

Do not use if allergic to silicone.

Storage

Store at room temperature, out of direct sunlight and keep dry.

Ordering information for Siltape®

Item#	Size	Packaging
CR3938	3/4" x 3 yds	1/bx, 12 bx/cs
CR3939	1-1/2" x 1.5 yds	1/bx, 12 bx/cs

Care and use symbols

-  Single use only
-  Keep dry
-  CE marked medical device
-  Store out of direct sunlight
-  Non-sterile



TOP TIP

Slit the Siltape® into the center using scissors to ensure dressing will seal well round nasogastric tube or drainage tube. This will help secure the tube.

Distributed by



DUKAL Corporation
2 Fleetwood Court
Ronkonkoma, NY 11779
Tel: 1-800-243-0741 | Email: advancisinfo@dukal.com

